


Patient HG Male Age: 19 years


History: HG was diagnosed with RE Pellucid marginal Degeneration and LE keratoconus 5 years ago. He wore RGP lenses but found them very uncomfortable with around 8 hours wearing time. Spectacles were not an option as he got no useful vision from them. This curtailed his life, particularly when trying to study.

Vision: R: 6/60 L: 1/60
Spectacle RX: R: Plano/-5.00 x 53 VA 6/18 pt L: -7.00/-6.00 x 145 VA 3/60
Keratometry: R: 7.90 x 65/7.03 x 155 L: 6.55 x 143/5.76 x 53

Topography (Absolute measurement):
RE

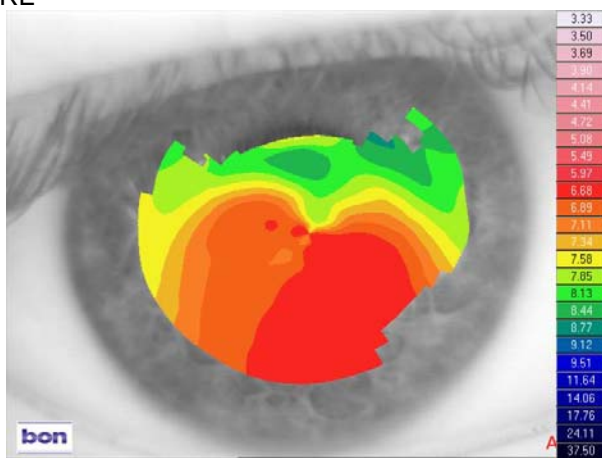


LE

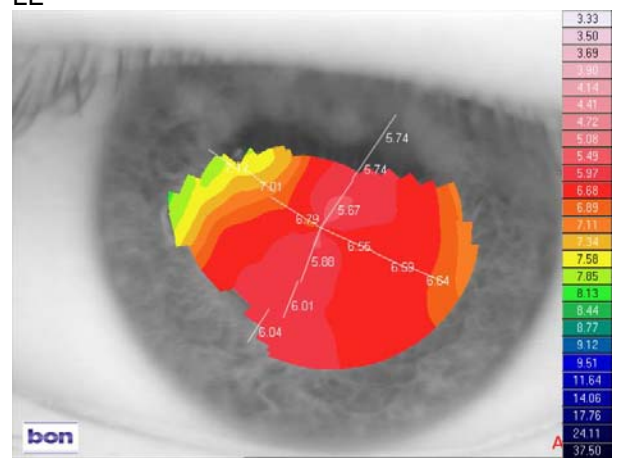


Elevation maps:

RE

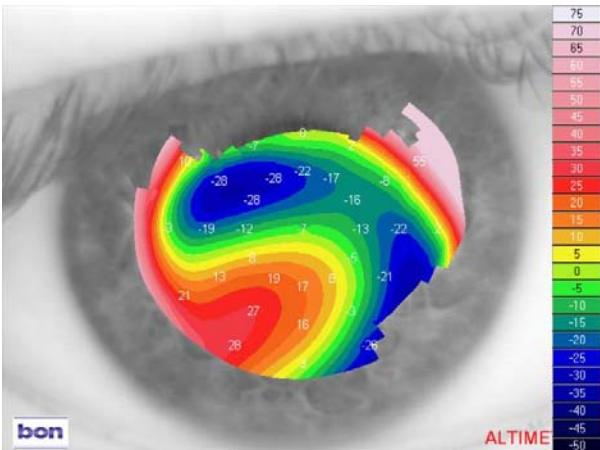

LE

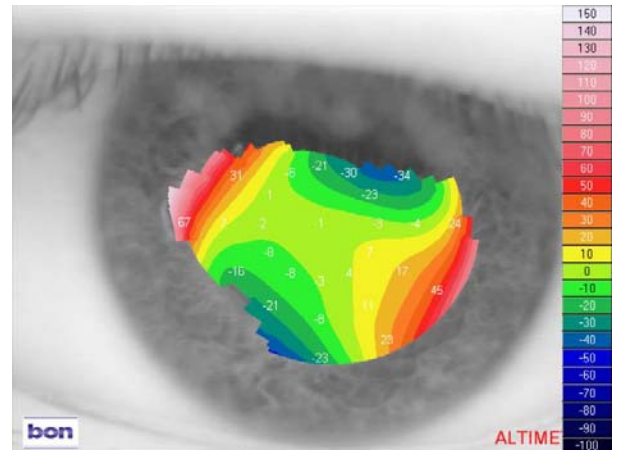

Lens type fitted: KeraSoft[®] 3 by UltraVision produced in Contamac Definitive™ Silicone Hydrogel material

R: 8.80/14.50/+0.50/-6.00 x 70 VA 6/6+2

L: 8.00/14.50/-7.00/-7.50 x 145 VA 6/6 +2

The left eye had been moulded by RGP wear and a month later, was refitted with:
L 8.20/14.50/-4.00/-7.00 x 130

On rechecking a month after this, the final lens was ordered:
L 8.20/14.0/-2.50/-8.00 x 130


UK Order Line: 0800 585115 (Freephone)

Tel: +44 (0)1525 381112 Fax: +44 (0)1525 370091


Email: orders@ultravision.co.uk Web: www.ultravision.co.uk; www.siliconehydrogel.co.uk; www.kerasoft3.com

ULTRAVISION INTERNATIONAL LIMITED, COMMERCE WAY, LEIGHTON BUZZARD, BEDFORDSHIRE, LU7 4RW, UNITED KINGDOM

Keratotomy with lenses in situ:
RE

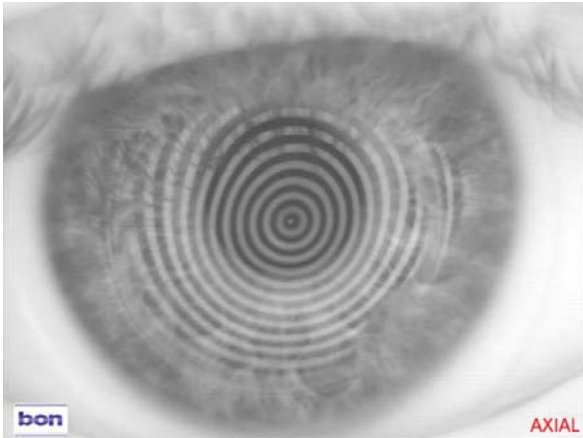

LE

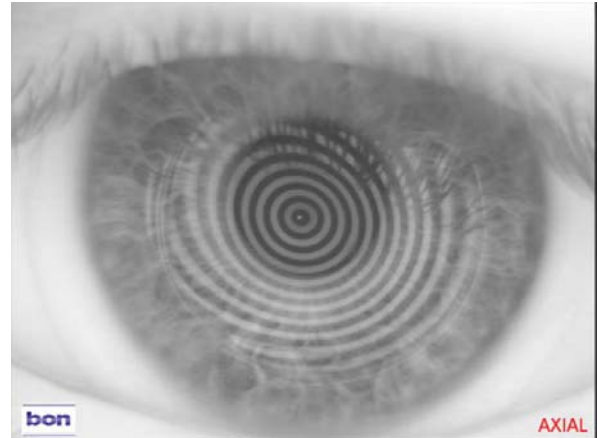

Wearing time: HG was able to wear the lenses for all waking hours which has made a tremendous difference to his lifestyle and enabled him to study much more effectively and improved his grades.

The left cornea in particular benefitted from wearing the KeraSoft[®]3 lens as it returned to its normal shape after discontinuing RGP wear.

UK Order Line: 0800 585115 (Freephone)

Tel: +44 (0)1525 381112 Fax: +44 (0)1525 370091

Email: orders@ultravision.co.uk Web: www.ultravision.co.uk; www.siliconehydrogel.co.uk; www.kerasoft3.com

ULTRAVISION INTERNATIONAL LIMITED, COMMERCE WAY, LEIGHTON BUZZARD, BEDFORDSHIRE, LU7 4RW, UNITED KINGDOM